

February 28, 2025

Notification of Routine Maintenance for OA-2000

Dear Valued Users,

We would like to inform you about the recommended routine maintenance for the OA-2000.

Due to the characteristics of laser light sources, prolonged usage can lead to aging, which may result in an increased possibility of measurement errors.

To reduce the risk of measurement errors, we kindly request that you:

- Always keep the software up to date.
- Perform maintenance by conducting a pre-use accessory model eye inspection.
- Conduct daily care and pre-use model eye inspections, regardless of the years in use.

We sincerely apologize for any inconvenience this may cause and kindly request your cooperation in conducting the device inspection. For assistance with software upgrades and more details, please contact our local distributor.

Thank you for your understanding and cooperation.

Sincerely,

International Department
TOMEY CORPORATION